Aqua Sonic

The Newest wonderful peeling for Skin Resurfacing



Technical Specifications

Ultrasonic	1.2 MHz
Motor	upto 600mmHg (±10%)
Power consumption	80Watt
Input Voltage	220VAC / 50 / 60Hz
Dimensions	350(D) X 350(L) X 1000 (H) mm
Weight	30Kg
Aqua Handpiece	Face(Regular Size) Nose(Small Size)
Aqua Waste tank	1200cc

Manufacturer : Ami

ADDRESS: RM. 1412, SJ TECHNO VILLE, 278, BEOTKKOT-RO, GEUMCHEON-GU, SEOUL, KOREA
Tel 82-2-3281-9091 Fax 82-2-3281-8011 E-mail amimed@amimed.co.kr

Skin Refreshing!

Make your skin and scalp cleaner and whiter.



Hydro Peeling System

Aqua Sonic

Remove horny substances and supply moisture at the same time.

Aqua sonic is designed to promote exfoliation to remove aged dead sin cells and sebum the same time, injects highly concentrated, nutritious serum to make skin and scalp moist, smooth, and lustrous.

Safe, no downtime, no side effect.

Because the treatment does not cause any damage to skin and scalp, patients can lead a normal life just after treatment. Also there is no side effect such as injury, pain, burns, rash, etc.

Multiple skincare effects.

Treatment removes foreign materials such as cosmetics and sebum.

Therefore, it is expected to achieve whitening and pore-tightening effects.



User Manual for Aqua Sonic

Aqua Sonic is ...

not a conventional decortication, but a new-concept natural skincare system.

Parts and components



Special tips

For removal of keratin as well as for extrusion







Around the nose


The special tip of the Aqua Sonic consists of two suction holes and one serum discharge hole.

its specially-designed spiral tip is intended to hold a product long enough to soften dead cells of the scalp and skin, provide nutrients, and absorb them in a vacuum state to maximize its peeling and extrusion effects.

Auga Sonic adopts a vacuum extrusion method. Therefore, the handpiece should be contacted to the skin closely. You can control pressure for both of fine bubbles and serum liquid to perform treatment. Red marks on the skin those may be created during operation will disappear normally In 20~30 minutes after treatment. And the skin will become healthy and lustrous condition.

01	Cleansing	Wipe wastes on the face 2+3 times with a cleansing cream
02	Diagnosis of scalp and skin and Selection of serum	Check the skin if normal, oily, sensitive, or of acne type and then select the serum suitable for the skin type.
03	Treatment [removal of keratin and sebum]	First spread the selected fine bubble and serum over the face and then divide the face into 3+5 areas to start treatment for about 2 minutes.
04	Washing	Wash and wipe out the face cleanly with a sponge after treatment.
05	Treatment (supply of moisture and nutrients)and finish	Treat to let serum penetrate into the skin; After treatment, give a finishing touch with a skin balm or a nourishing cream.

How to Treat? Recommended treatment is kept on around 2 minutes in the order of;

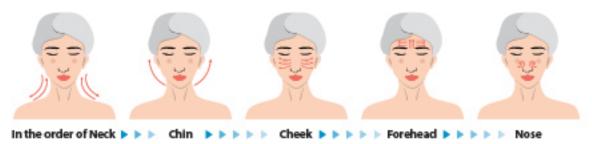

Neck: Turning face a little to the right and left, make the lymph nodes relax by massaging the neck up to down and down to up.

Chin: Massage the chin area from bottom to upper part, not to the opposite

Cheek: Massage this part from nose to outside.

Forehead: Massage Forehead down to up and in to out direction.

A lot of sebum is near the nose. Therefore, more massages are needed for this area.

Massage both sides of the nose making small circles.

Full Face: Finally massage across the full face from bottom to top smoothly.

Aqua Sonic Serum

AQUA PI 200

AQUA PE



Dr.Serum

AQUA PE [For all Skin Type-For the general skin, 30ml]

Since glucosamine and lactic acids promote the peeling of the skin to remove impure substances softly and plant extracts soothe and moisten the skin smoothly, this product enables a scientific treatment that does not cause any injuries or rashes while achieving a bright and healthylooking skin regeneration.

[Nutrition and supply of moisture for all types of skin, 20ml & 490ml]

The deep penetration of vitamin C.E. Aloe Vera, and green tea extract will help skin to restore and regenerate damaged skin cells to maintain a moist and bright skin tone without any side